

**Safety Communication**

**رسالة سلامة**

**ABP devices shutting down while running on battery power**

<b>Device/ Product Description:</b>	Intra-Aortic Balloon Pumps
<b>Brand:</b>	Cardiosave (Hybrid and Rescue), CS300 and CS100/CS100i
<b>Lot/Serial Number:</b>	ALL
<b>Manufacturer:</b>	Getinge's Maquet/Datascope
<b>Problem:</b>	The devices may shut down while running on battery power, leading to pump stop and loss of hemodynamic support. An interruption of the therapy would be unlikely to lead to a life-threatening situation. However, in critical patients with severely compromised hemodynamic function dependent on continuous circulatory support, an interruption or delay in Intra-Aortic Balloon support as a result of an unexpected shutdown or failure to initiate therapy can occasionally/likely have more severe consequences that can be life threatening.
<b>Recommendation/Actions:</b>	<ol style="list-style-type: none"> <li>1- review the firm's <a href="#">notice</a>, and the developed quick reference <a href="#">guides</a></li> <li>2- Follow each device's Operating Instructions Manual</li> <li>3- Ensure that, whenever possible, the device is plugged into an AC power outlet (during patient use, and when the system is not in use)</li> <li>4- Refer to the Operating Instructions Manual for recommendations on portable/battery operation when transporting patients.</li> <li>5- Check battery run time and replace batteries as required, as recommended in the Operating Instructions Manual.</li> </ol>

6- Replace batteries for the “Cardiosave” devices that are older than 4 years with only Datascope approved/sourced batteries.

7- Follow informational messages on the display screen for CS100/CS300 devices regarding the batteries.

8- [Report](#) to the SFDA events of devices shutting down while running on batteries.

9- Contact the authorized representative to provide the required support.

For more information, Please click [here](#).

**Devices/Products photo:**



<b>Authorized Representative Details</b>	AR name:	Al-Jeel Medical
	Assigned Contact Person:	Meshal Aleshaiwy
	Mobile/Phone:	0508827808
	Email:	<a href="mailto:aljeel.sfda@aljeel.com">aljeel.sfda@aljeel.com</a>