


**Safety Communication**

**رسالة سلامة**

**Contact between electrode and defibrillator may be interrupted**

<b>Device/ Product Description:</b>	Defibrillation electrodes
<b>Brand:</b>	corPatch easy
<b>REF Number:</b>	04324.3
<b>Lot Number:</b>	190711-3993
<b>Manufacturer:</b>	Leonhard Lang GmbH
<b>Problem:</b>	<p>It was discovered that there is a possible contact problem between electrode and ring terminal in the mentioned lot number. The contact between electrode and defibrillator may be interrupted or partially interrupted.</p> <p>There is a potential risk that no signals or energy will be transmitted. This can result in failure to treat patient in a life-threatening condition requiring a defibrillation shock on time.</p>
<b>Recommendation/Actions:</b>	<ol style="list-style-type: none"><li>1- inventory the electrodes stock in your facility, and isolate the affected products immediately</li><li>2- If you found that the affected electrode is already in use, ensure to have at least on spare pair in reserve until the electrodes have been replaced.</li><li>3- Contact the Authorized representative for support and replacement</li></ol> <p>for more information, please click:</p> <p> Vk_20191219_20-EN-1 (Leonhard Lang GmbH).pdf</p>

<b>Devices/Products photo:</b>		
<b>Authorized Representative Details</b>	AR name:	Teriag International Trading Company
	Assigned Contact Person:	Syed Naimuddin
	Mobile/Phone:	0502474817
	Email:	info@teriag.com

