

Safety Communication

رسالة سلامة

Defibrillator may not deliver a shock after pressing the "Shock" button on the keypad

Device/ Product Description:	Monitor/Defibrillator
Brand:	LIFEPAK® 15
Part Number:	V15-2-XXXXXX
Manufacturer:	Stryker
Problem:	The device may not deliver a defibrillation shock when the device "Shock" button is pressed as a result of oxidation that has formed over time within the button. The hard paddle shock button is not affected by this issue.
Recommendation/Actions:	<ol style="list-style-type: none">1. Review this notice and ensure that affected personnel are aware of the contents.2. Perform the daily check as described in the Operator's Checklist to detect the problem prior to patient use.3. If you have affected product at your facility, contact the authorized representative to arrange for the correction of your device. For more information, Please click here .

Devices/Products photo:



Authorized Representative Details

AR name:	Aljeel Medical
Assigned Contact Person:	Meshal Aleshaiwy
Mobile/Phone:	0508827808
Email:	aljeel.sfda@aljeel.com

SFDA