

Safety Communication

رسالة سلامة

Risk of a false positive result when using the Reagents for the BD MAX™ System.

Device/ Product Description:	SARS-CoV-2 Reagents
Brand:	BD MAX™ System
Affected product:	REFs: 445003 and 445003-01
Manufacturer:	Becton Dickinson (BD)
Problem:	<p>The manufacturer has identified a potential for false positive results when using the Reagents for the BD MAX™ System.</p> <p>A false positive SARS-CoV-2 test result could lead to a delay in diagnosis and potentially cause the patient to be exposed to COVID-19 if they are isolated with an infected individual.</p>
Recommendation /Actions:	<p>Recommendations for Health Care Providers and Facility Staff:</p> <ul style="list-style-type: none">• Make sure that this document, with accompanying FSNs, are reached to the end-users.• The user Defined Protocol (UDP) threshold for the N2 target in the IFU needs to be revised from 50 RFU to 80 RFU .• Consider any positive result presumptive from tests using the BD SARS-CoV-2 Reagents for the BD Max System. Consider confirming with an alternate authorized test. <p>For more information, Please click here.</p> <p>If you think you had a problem with your device or a device your patient uses, please do not hesitate to report the problem to SFDA through: NCMDR Vigilance system</p>

	19999 unified call center	
Devices/Products photo:		
Authorized Representative Details	AR name:	Becton Dickinson B.V.
	Assigned Contact Person:	Saleh Al sohibanei
	Mobile/Phone:	0544760333
	Email:	Saleh.AL.SOHIBANEI@bd.com

