

## Safety Communication

## رسالة سلامة

### A number of bugs and technical issues

<b>Device/ Product Description:</b>	blood purification device
<b>Brand Name:</b>	HF440
<b>Affected product:</b>	Software Release Versions: 2.8 to 3.1.4
<b>Manufacturer:</b>	Infomed SA
<b>Problem:</b>	<p>A number of bugs and technical issues corrected through software updates include:</p> <ul style="list-style-type: none"> <li>- Waste Flow Errors: Leads to interrupted treatment and erroneous error codes.</li> <li>- Wi-Fi Connection: Wi-Fi being turned on can crash unit during treatment.</li> <li>- Treatment Data: Treatment history, nursing staff record data to patient records not available.</li> <li>- Date Format Conflicts: Date format inconsistent across different screens; sometimes DD/MM/YYYY, other times MM/DD/YYYY</li> <li>- Intermittent screen freezing: Touchscreen becomes inoperable.</li> </ul>
<b>Recommendation /Actions:</b>	<ul style="list-style-type: none"> <li>• Make sure that this document is reached to the end-users.</li> <li>• Contact the authorized representative for the required software updates.</li> </ul> <p>For more information, Please see click <a href="#">here</a>.</p> <p>If you think you had a problem with your device or a device your patient uses, please report the problem to SFDA through:</p>

NCMDR  
Vigilance system  
 (19999) unified call center

**Devices/Products photo:**



<b>Authorized Representative Details</b>	AR name:	Advance Nfood Medical Company
	Assigned Contact Person:	Abdullah al dosary
	Mobile/Phone:	+(966) 501455589
	Email:	<a href="mailto:fayaz@nfood.com.sa">fayaz@nfood.com.sa</a>