

Safety Communication

رسالة سلامة

Insulin pumps-Reservoir May Become Loose, Potentially Preventing

Device/ Product Description:	MiniMed 600 series insulin pumps	
Affected product:	MiniMed 600 series insulin pumps with a clear retainer ring	
Manufacturer:	Medtronic SA	
Problem:	The MiniMed 600 series insulin pump is designed with a pump retainer ring to lock the reservoir in the insulin pump. There have been reported incidents of a loose reservoir that can no longer be locked into the pump. The reservoir can become loose due to a broken or missing retainer ring that prevents a proper lock.	
Recommendation /Actions:	Medtronic will replace any MiniMed 600 series insulin pump that has a clear retainer ring with a MiniMed 600 series insulin pump that has the updated black retainer ring design. For more information, please click here . If you think you had a problem with your device or a device your patient uses, please report the problem to SFDA through: NCMDR Vigilance system (19999)unified call center	
Authorized Representative Details	AR name:	Medtronic Saudi Arabia
	Assigned Contact Person:	Faisal Matbuli
	Mobile/Phone:	+(966) 535355500
	Email:	ksa.ra@medtronic.com