

Safety Communication

رسالة سلامة

Risk of Strangulation in Children who Use Enteral Feeding Delivery Sets

Device/ Product Description:	Enteral Feeding Delivery Sets
Affected product:	All
Problem:	There is a risk of strangulation from the use of enteral feeding delivery sets. The feeding set tubing can become wrapped around a child's neck and cause strangulation or death.
Recommendation /Actions:	<ul style="list-style-type: none"> Review this topic and the information your colleagues, care teams, and caregivers of pediatric patients who use enteral feeding delivery sets, to ensure they are aware of the potential risk of strangulation with the associated tubing and are taking appropriate measures to keep the tubing away from the child as much as possible. When caring for pediatric patients who receive enteral feeding and as part of an individual risk assessment, be aware of the risk of strangulation from the feeding set tubing and follow protocols to monitor medical line safety. To the extent possible, avoid leaving the feeding set tubing where infants or children can become entangled. <p>For more information, please click here.</p> <p>If you think you had a problem with your device or a device your patient uses, please report the problem to SFDA through: NCMDR Vigilance system (19999) unified call center</p>