

Date: 21-03-2023




Reference Number: SG-2303-404-H

قطاع الأجهزة والمستلزمات الطبية  
المركز الوطني لبلاغات الأجهزة والمستلزمات الطبية

Medical Devices Sector  
National Center for Medical Devices Reporting

الهيئة العامة للغذاء والدواء  
Saudi Food & Drug Authority

رسالة سلامة  
Safety Communication

To: Healthcare Provider		إلى: مقدمي الرعاية الصحية
Title	Safe Usage of IV Cannula	العنوان
Medical Device Description	IV cannula	اسم ووصف الجهاز/المستلزم الطبي
Manufacturer	Saudi Mais Co. for medical Products (SMMP)	اسم المصنع
Medical Devices Marketing Authorization (MDMA)	MDMA-1-2017-2109	إذن التسويق
Potential /Associated risks:	Breakage of Catheter (FEP tube) Leakage in Catheter (FEP tube)	المخاطر المحتملة/ المرتبطة بالجهاز أو المستلزم الطبي
Recommendations	<b>Withdrawal of needle:</b> when successful insertion is achieved, withdraw the needle gently, to prevent from surpassing the posterior vein wall, then advance the catheter into the vein & immediately release the tourniquet. <b>Do not attempt to reinsert</b> a partially or completely withdrawn needle. Withdraw the needle completely & never attempt to re-insert the needle, it may damage the catheter (FEP tube).	التوصيات
For Reporting	  	للإبلاغ: